# VertiSVV®

Subjective Visual Vertical  
Subjektive Visuelle Vertikale

## Installation instructions

---

## Installationsanleitung

*Life needs balance!*

**KM-ENKE**  
GmbH

**ZEHNIT®**

### 1.1 What is VertiSVV?

VertiSVV is a system that lets you perform the Subjective Visual Vertical (SVV) and Subjective Visual Horizontal (SVH) tests to detect otolith and central disorders. It consists of three components:

- » Virtual-reality goggles that eliminates any visual references for the subject.
- » A specially designed, easy-to-use wireless controller.
- » The VertiPACS software, which leads the user through the tests, analyses the data (mean, standard deviation, etc.) and enables editing and visualising the test results, including normative data.

### 1.2 Intended user

The VertiSVV system is intended to be used only by qualified medical personnel.

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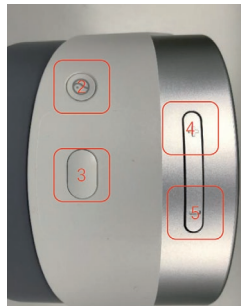
**2.1 Button description**

- 1) **Power-button**  
press and hold for 5 seconds to stop power on \ power off and briefly press to put the device into dormancy.
- 2) **Home-button**  
To go back to the previous menu, double-click this button.
- 3) **Confirmation-button**  
click to confirm to enter the lower page or adjust the relevant functions.
- 4) **Up-button (volume + button)**  
press this button to move up one step, or to increase the volume.
- 5) **Down-button (volume - button)**  
press this button to move down one step, or to decrease the volume.
- 6) **3.5 mm headphone port.**
- 7) Charging port and SW controller receiver port.

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### 3.1 Presets for the Bluetooth pairing

- » Switch on VertiSW.
- » Put on your VertiSW.
- » Press and hold the Power-button for 5 seconds (1).  
If VertiSW has started or is in dormancy, briefly press the Power-button (1).
- » Wait until the system has booted.

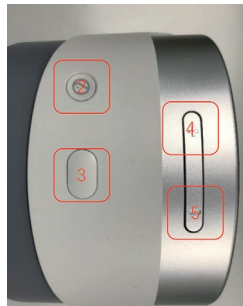
### 3.2 Bluetooth settings

To open the Bluetooth settings, use the key combination.

Press the Confirmation-button (3) and the Down-button (5) simultaneously to enter the Bluetooth settings page to synchronize the device with your PC.

You will see a circled cursor on the screen of your VertiSW.

- » Navigate the cursor by moving your head.
- » Hover over "Bluetooth" with the cursor and press the Confirmation-button (3).

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
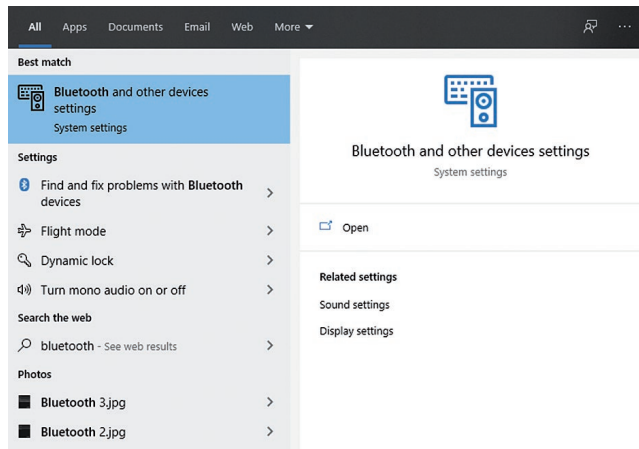
## VertiSVV® 4 Computer settings/Computer-Einstellungen

- » Enter "Bluetooth" in the input field in the lower left corner of your screen.
- » Click on "Bluetooth and other device settings".

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
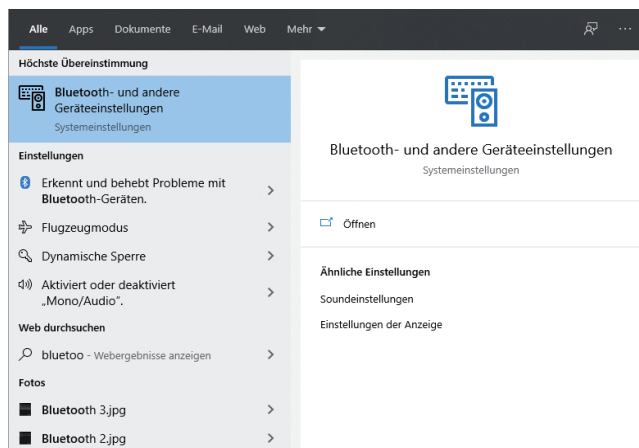
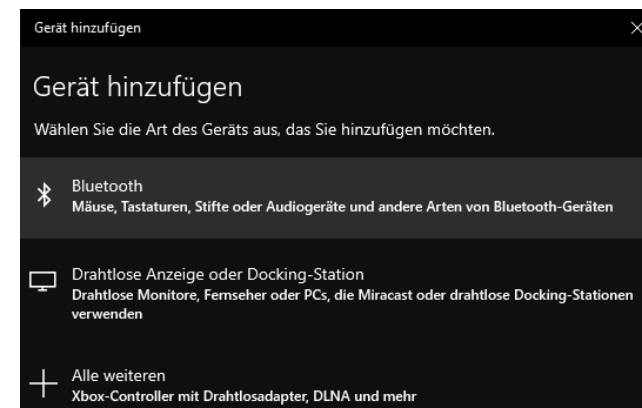
## VertiSVV® 4 Computer settings/Computer-Einstellungen

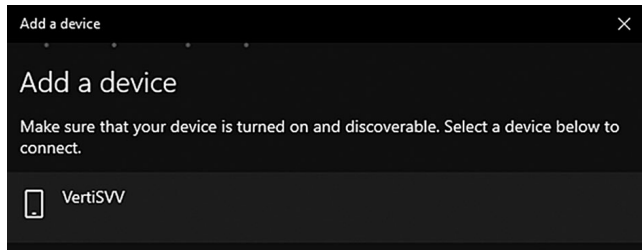- » Choose VertiSW.

Watch the display in your device to see if a PIN is shown. Click on the Confirmation-button (3) to confirm this PIN if it matches the one on your computer.

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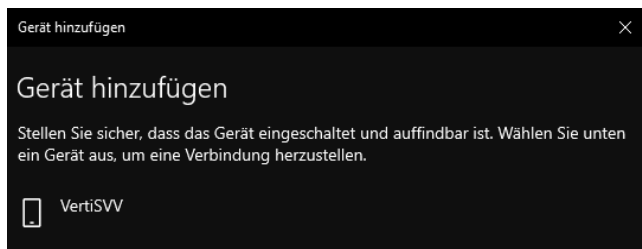
## VertiSVV® 5 VertiPACS settings/VertiPACS-Einstellungen

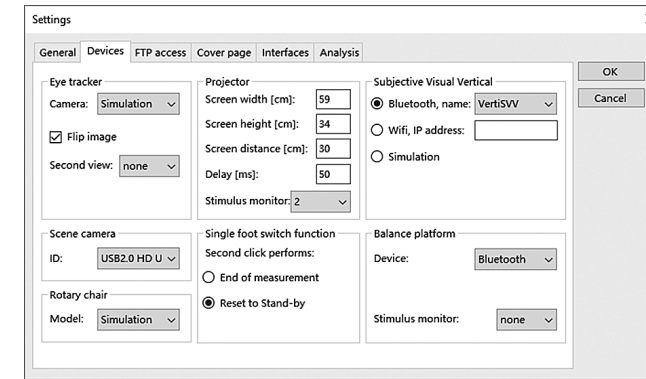- » Open VertiPACS on your PC and go to Settings – Program – Devices.
- » Select „Bluetooth, name:“ and choose VertiSVV.
- » Confirm with OK.

For instructions on how to use the VertiPACS program module „Virtual Subjective Vertical“ please refer to the VertiPACS manual.

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KM-ENKE GmbH · Monreposstr. 49